# RadiForce® MX220W/MX241W

## Color LCD Monitor

English

Deutsch

Français

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### Instructions for Use: ■ Setup Manual

**Important:** Please read this Setup Manual and the User's Manual stored (on the CD-ROM) carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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Color LCD Monitor

## SAFETY SYMBOLS

This manual uses the safety symbols below. They denote critical information. Please read them carefully.

 <p><b>WARNING</b> Failure to abide by the information in a WARNING may result in serious injury and can be life threatening.</p>	 <p><b>CAUTION</b> Failure to abide by the information in a CAUTION may result in moderate injury and/or property or product damage.</p>
	<p>Indicates a prohibited action.</p>
	<p>Indicates a mandatory action that must be followed. For example, the symbol  illustrates the notification of general prohibition such as "Grounding the unit".</p>

## PRECAUTIONS

### WARNING

**If the unit begins to emit smoke, smells like something is burning, or makes strange noises, disconnect all power connections immediately and contact your dealer for advice.**

Attempting to use a malfunctioning unit may result in fire, electric shock, or equipment damage.

**Use the enclosed power cord and connect to the standard power outlet of your country.**

Be sure to remain within the rated voltage of the power cord. Not doing so may result in fire or electric shock.

"Power Supply: 100-120/200-240 Vac 50/60Hz"

**Keep small objects or liquids away from the unit.**

Small objects accidentally falling through the ventilation slots into the cabinet or spillage into the cabinet may result in fire, electric shock, or equipment damage. If an object or liquid falls/spills into the cabinet, unplug the unit immediately. Have the unit checked by a qualified service engineer before using it again.

**Use the unit in an appropriate location.**

Not doing so may result in fire, electric shock, or equipment damage.

- Do not place outdoors.
- Do not place in the transportation system (ship, aircraft, trains, automobiles, etc.).
- Do not place in a dusty or humid environment.
- Do not place in a location where water is splashed on the screen (bathroom, kitchen, etc.)
- Do not place in a location where the steam comes directly on the screen.
- Do not place near heat generating devices or a humidifier.
- Do not place in a location where the product is subject to direct sunlight.
- Do not place in an inflammable gas environment.

**The equipment must be connected to a grounded main outlet.**

Not doing so may result in fire or electric shock.

**Fluorescent backlight lamps contain mercury (the products that have LED backlight lamps contain no mercury), dispose according to local, state or federal laws.**

Exposure to elemental mercury can result in effects on the nervous system, including tremor, memory loss, and headache.

### CAUTION

**Do not block the ventilation slots on the cabinet.**

- Do not place any objects on the ventilation slots.
- Do not install the unit in a closed space.
- Do not use the unit laid down or upside down.

Blocking the ventilation slots prevents proper airflow and may result in fire, electric shock, or equipment damage.

**Use an easily accessible power outlet.**

This will ensure that you can disconnect the power quickly in case of a problem.


## Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.


### NOTE

Please keep the packaging box and materials for future movement or transport of the monitor.


- Monitor
- Power cord
- Digital signal cable : PP300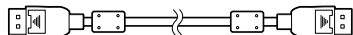

- Digital signal cable : DD300

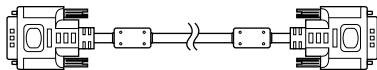
- EIZO USB cable : UU300

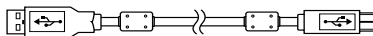- EIZO LCD Utility Disk (CD-ROM)
  - RadiCS LE (for Windows)
  - ScreenManager Pro for Medical (for Windows)
  - User's Manual
- Setup Manual (this manual)
- Recycling Information

## Controls and Functions

Ex. MX241W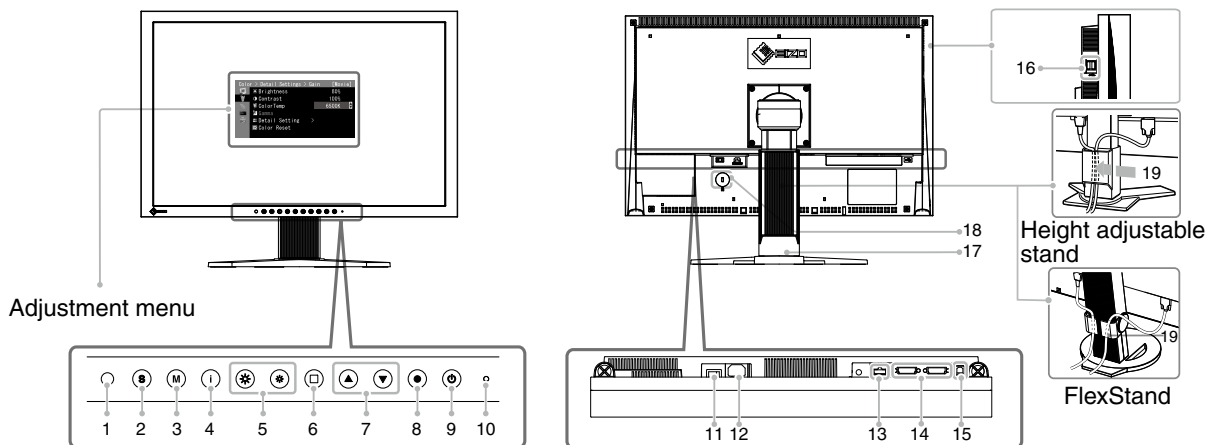

1	<b>Sensor</b>	Detects ambient brightness. Auto EcoView function.
2	<b>(S) button</b>	Switches input signals for display when more than two PCs are connected to the monitor.
3	<b>(M) button</b>	Allows you to switch the CAL Switch mode (page 5).
4	<b>(i) button</b>	Displays the information about monitor or input signals.
5	<b>(*)* button</b>	Adjusts the brightness.
6	<b>(□) button</b>	Cancels the adjustment/setting or exit the adjustment menu.
7	<b>(▲)(▼) button</b>	Chooses an adjustment item or increases/decreases adjusted values for advanced adjustments using the Adjustment menu.
8	<b>(●) button</b>	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
9	<b>(⏻) button</b>	Turns the power on or off.
10	<b>Power indicator</b>	Indicates monitor's operation status. Blue: Operating Orange: Power saving Off: Main Power Switch / Power button off
11	<b>Main Power switch</b>	Turns the main power on or off.
12	<b>Power connector</b>	Connects the power cord.
13	<b>Input signal connectors</b>	DisplayPort Connector x 1
14	<b>Input signal connectors</b>	DVI-I Connector x 2
15	<b>USB port (Up)</b>	Connects the USB cable to use the software that needs USB connection, or to use USB Hub function.
16	<b>USB port (Down)</b>	Connects a peripheral USB device.
17	<b>Stand</b>	Used to adjust the height and angle of the monitor screen.
18	<b>Security lock slot</b>	Complies with Kensington's MicroSaver security system.
19	<b>Cable holder</b>	Covers the monitor cables.

# Connecting Cables

## Note


- Check that the monitor and the PC are powered off.
- When replacing the current monitor with a MX220W/MX241W monitor, be sure to change the PC settings for resolution and vertical frequency to those which are available for the MX220W/MX241W monitor referring to the resolution table (back of cover page) before connecting the PC.

## Tips

- When connecting more than two PCs to the monitor, refer to the User's Manual on the CD-ROM.

## 1 Connect the monitor to the PC with a signal cable that matches the connectors.

After connecting the cable connectors, tighten the screws of the connectors to secure the coupling.

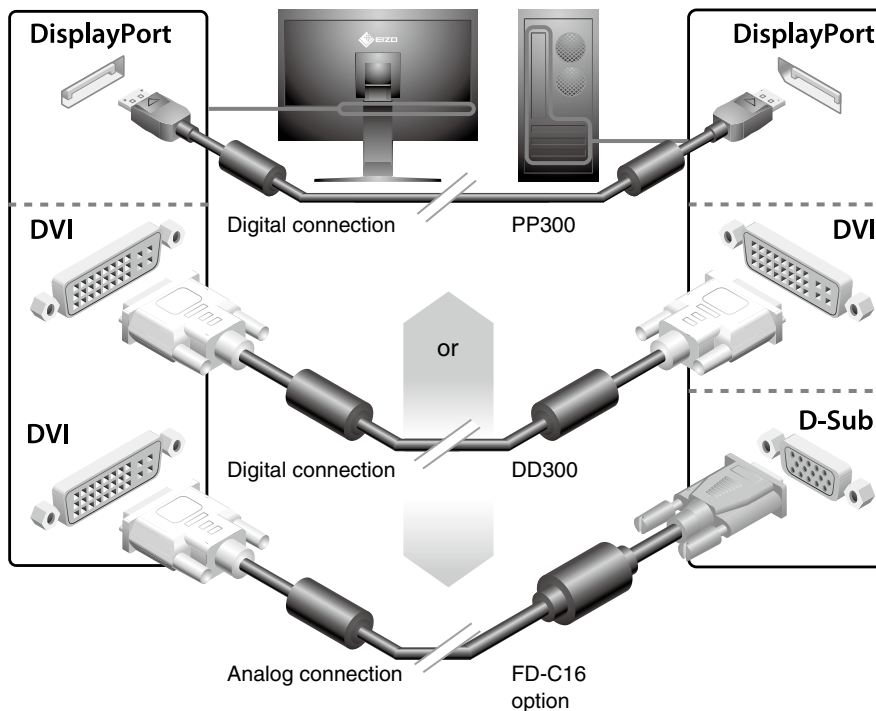## 2 Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.

Connecting to the Monitor

Connecting to a Computer

## 3 Plug the power cord into a power outlet and the power connector on the monitor.

## 4 Press to turn on the monitor.

The monitor's power indicator lights up blue.

## 5 Turn on the PC.

The screen image appears.

When turning on the monitor and PC for the first time with the analog signal, the Self Adjust function works to adjust the clock, phase, and display position automatically.


If an image does not appear, refer to "No-Picture Problem" (page.6) for additional advice.

## Note

- Turn off the monitor and PC after using them.
- For the maximum power saving, it is recommended that the Power button be turned off. Turning off the main power switch completely shuts off power supply to the monitor.

# Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.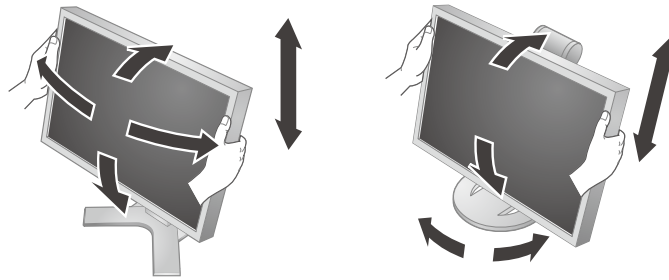

## Note

- Be sure that the cables are correctly connected.

# Installing RadiCS LE

The quality control software "RadiCS LE" (for Windows) is included with MX series monitors. To calibrate the monitor and manage the calibration history, install RadiCS LE with reference to the RadiCS LE User's Manual on the CD-ROM.

## Note


- Connect the monitor to the PC with a USB cable when using RadiCS LE.

## Tips

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit "RadiCS UX1".

# Showing Button Guide

Press the front buttons (except ☺ button), the button guide appears above the button. (When using the monitor with "Portrait" position, the button guide appears next to the button.)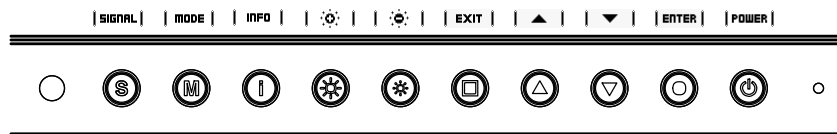

## Tips


- The button guide will continue to appear while the Adjustment menu or Mode menu is showing.
- The button guide is displayed differently depending on the selected menu or status.

# Selecting Display Mode

CAL Switch allows you to select easily the adequate mode suitable for the monitor's application.

## CAL Switch Mode

<b>1-DICOM</b>	Available for setting tailored to X-ray film color (clear base).
<b>2-Custom</b>	Available for the color settings according to your preference.
<b>3-CAL</b>	Displays the screen adjusted by calibration software.
<b>4-Text</b>	Suitable for displaying texts for word processing or spreadsheets.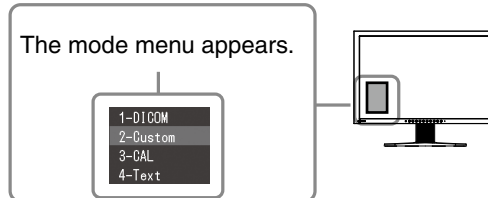


### 1 Press **(M)**.

Mode menu appears at the lower left of the screen.

### 2 The mode among the list is highlighted in turn each time **(M)** is pressed.

You can switch the mode with **(▲)** or **(▼)** while the mode menu is displayed.

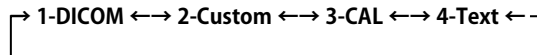The basic settings and adjustments are completed. For advanced settings/adjustments, refer to the User's Manual on the CD-ROM.

# No-Picture Problem



If no picture is displayed on the monitor even after the following remedial action is taken, contact your local dealer.

## 1. Check the Power indicator.

Symptom	Status	Possible cause and remedy
No picture	Power indicator does not light up.	<ul style="list-style-type: none"> <li>• Check whether the power cord is connected correctly.</li> <li>• Turn off the main power, and then turn it on again a few minutes later.</li> <li>• Turn the main power switch on.</li> <li>• Press .</li> </ul>
	Power indicator lights blue.	<ul style="list-style-type: none"> <li>• Set each adjusting value in [Brightness], [Contrast] and [Gain] to higher level.</li> </ul>
	Power indicator lights orange.	<ul style="list-style-type: none"> <li>• Switch the input signal with .</li> <li>• Press a key on the keyboard or click the mouse.</li> <li>• Turn on the PC.</li> </ul>

## 2. Check the error message that remains on the screen.

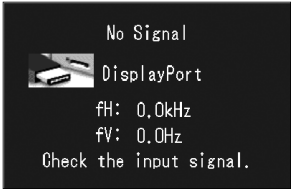These messages appear when the input signal is incorrect even if the monitor is functioning.

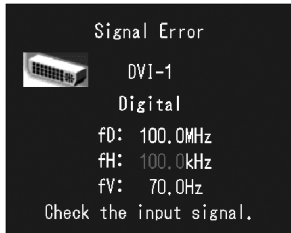Symptom	Status	Possible cause and remedy
 <p>No Signal   DisplayPort            fH: 0.0kHz            fV: 0.0Hz            Check the input signal.</p>	The input signal is not received.	<ul style="list-style-type: none"> <li>• Check whether the PC is turned on.</li> <li>• Check whether the signal cable is properly connected to the PC.</li> <li>• Switch the input signal with .</li> </ul>
 <p>Signal Error   DVI-1            Digital            fD: 100.0MHz            fH: 100.0kHz            fV: 70.0Hz            Check the input signal.</p>	The signal frequency is outside the specification. Incorrect signal frequency is shown in red.	<ul style="list-style-type: none"> <li>• Reboot the PC.</li> <li>• Change the mode to an appropriate mode using the graphics board's utility software. Refer to the manual of the graphics board for details.</li> </ul> fD: Dot Clock (Displayed only when the digital signal inputs) fH: Horizontal Frequency fV: Vertical Frequency

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---

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.

EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

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<http://www.eizo.com>

Copyright © 2010-2014 EIZO Corporation All rights reserved.

7th Edition-January, 2014 Printed in Japan.

00N0L629G1  
(U.M-SUMX241W-C-4)

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## **EIZO GmbH**