Instructions for Use

■ User's Manual

RadiForce® MX215

Color LCD Monitor

Important

Please read PRECAUTIONS, this User's Manual, and the Setup Manual (separate volume) carefully to familiarize yourself with safe and effective usage.

-
- Please refer to the Setup Manual for basic information ranging from connection of the monitor to a PC to using the monitor.
 - The latest User's Manual is available for download from our web site:
<http://www.eizoglobal.com>
-

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

Notice for this monitor

This product is intended to be used in displaying and viewing digital images for review and analysis by trained medical practitioners. It does not support the display of mammography images for diagnosis.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

This product has been adjusted specifically for use in the region to which it was originally shipped. If the product is used outside the region, it may not operate as specified in the specifications.

This product may not be covered by warranty for uses other than those described in this manual.

The specifications noted in this manual are only applicable when the following are used:

- Power cords provided with the product
 - Signal cables specified by us
-

Only use optional products manufactured or specified by us with this product.

It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.

Monitors should be set to a lower brightness to reduce changes in luminosity caused by long-term use and maintain a stable display.

When the screen image is changed after displaying the same image for extended periods of time, an afterimage may appear. Use the screen saver or power save function to avoid displaying the same image for extended periods of time.

Periodic cleaning is recommended to keep the monitor looking new and to prolong its operation lifetime (refer to [“Cleaning” \(page 4\)](#)).

The screen may have defective pixels or a small number of light dots on the screen. This is due to the characteristics of the panel itself, and is not a malfunction of the product.

The backlight of the LCD panel has a fixed lifetime. When the screen becomes dark or begins to flicker, please contact your local EIZO representative.

Do not press on the panel or edge of the frame strongly, as this may result in display malfunctions, such as interference patterns, etc. If pressure is continually applied to the panel, it may deteriorate or damage your panel. (If the pressure marks remain on the panel, leave the monitor with a black or white screen. The symptom may disappear.)

Do not scratch or press on the panel with any sharp objects, as this may result in damage to the panel. Do not attempt to brush with tissues as this may scratch the panel.

When the monitor is cold and brought into a room or the room temperature goes up quickly, dew condensation may occur on the interior and exterior surfaces of the monitor. In that case, do not turn the monitor on. Instead wait until the dew condensation disappears, otherwise it may cause some damage to the monitor.

For users in the territory of the EEA and Switzerland:

Any serious incident that has occurred in relation to the device should be reported to the Manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

Cleaning

Attention

- Chemicals such as alcohol and antiseptic solution may cause gloss variation, tarnishing, and fading of the cabinet or panel, and also quality deterioration of the image.
- Never use any thinner, benzene, wax, and abrasive cleaner, which may damage the cabinet or panel.

Note

- The optional ScreenCleaner is recommended for cleaning the cabinet and panel surface.

The stains on the cabinet and panel surface can be removed by moistening part of a soft cloth with water.

To use the monitor comfortably

- An excessively dark or bright screen may affect your eyes. Adjust the brightness of the monitor according to the environmental conditions.
- Staring at the monitor for a long time tires your eyes. Take a 10-minute rest every hour.

CONTENTS

Notice for this monitor	3	Chapter 4 Controlling Monitor Quality	25
Cleaning	4	4-1. Executing Calibration	25
To use the monitor comfortably	4	● Executing SelfCalibration	25
CONTENTS	5	● Viewing SelfCalibration result	26
Chapter 1 Introduction	7	4-2. Checking Grayscale	26
1-1. Features	7	● Executing Grayscale Check	26
1-2. EIZO LCD Utility Disk	8	● Viewing grayscale check results	26
● Disk contents and software overview	8	4-3. Acquiring Life Prediction Data	27
● Using RadiCS LE/ScreenManager Pro for Medical	8	4-4. Setting On/Off for Warning and QC History	27
1-3. Basic Operations and Functions	9	Chapter 5 Power Saving Functions	28
● Basic Adjustment menu operations	9	5-1. Setting Power Save	28
● Functions	10	5-2. Adjusting the Brightness of Indicator	28
Chapter 2 Adjusting Screens	12	5-3. Setting DisplayPort Power Save	29
2-1. Compatible Resolutions	12	5-4. Setting the Power Saving Mode when a Person Leaves the Monitor	30
2-2. Setting the Resolution	12	Chapter 6 Troubleshooting	31
● Windows 8 / Windows 7	12	6-1. No Picture	31
● Windows Vista	12	6-2. Imaging Problems (for both digital and analog)	32
● Windows XP	12	6-3. Imaging Problems (for analog only)	32
● Mac OS X	12	6-4. Other Problems	33
2-3. Adjusting Color	13	● Error code table	34
● Selecting the display mode (CAL Switch mode)	13	Chapter 7 Reference	35
● Performing advanced adjustments	13	7-1. Attaching Arm	35
● Adjustable settings in each mode	14	7-2. Connecting Multiple PCs	36
● Adjusting Brightness	14	● Switching among input signals	36
● Adjusting Temperature	14	● Setting input signal switching procedure	37
● Adjusting Gamma	15	7-3. Making Use of USB (Universal Serial Bus)	38
● Adjusting Hue	15	● Required system environment	38
● Adjusting Saturation	15	● Connection procedure (Setup of USB function)	38
● Adjusting Gain	16	7-4. Showing Monitor Information	39
2-4. Adjusting Screens	17	● Displaying signal information	39
● Digital Signal Input	17	● Showing monitor information	39
● Analog Signal Input	17	7-5. Specifications	40
2-5. Selecting Screen Size	21	7-6. Preset Timing	43
Chapter 3 Setting Monitor	22	Chapter 8 Glossary	44
3-1. Setting Orientation	22	Appendix	46
3-2. Enabling/Disabling Mode Selection	22	Trademark	46
3-3. Setting Language	23	License	46
3-4. Setting Adjustment Menu Display	23	Medical Standards	47
3-5. Showing and hiding the EIZO logo	23	EMC Information	48
3-6. Locking Operation Buttons	24	FCC Declaration of Conformity	53
3-7. Resetting to Default	24		
● Resetting the color adjustments	24		
● Resetting all adjustments to the default settings	24		

LIMITED WARRANTY..... 54

Recycling Information 62

Chapter 1 Introduction

Thank you very much for choosing an EIZO color LCD monitor.

1-1. Features

- 21.3 inches
- Supports a resolution of 2M pixels (Portrait: 1200 × 1600 dots (H × V))
- Uses high-contrast panel (1500:1).
Enables the display of sharp images.
- Applicable to DisplayPort (applicable to 8 bit or 10 bit, not applicable to audio signals)
- The RadiCS SelfQC function and the built-in integrated front sensor allow the user to perform the calibration and Grayscale Check for the monitor independently.
See [“Chapter 4 Controlling Monitor Quality” \(page 25\)](#).

Attention

- When a panel protector is attached, the Integrated Front Sensor cannot be used.
 - A high temperature or high humidity environment may affect the measurement accuracy of the integrated front sensor. We suggest storing and using the monitor under the following conditions.
 - - Temperature 30°C or less
 - - Humidity 70% or less
 - Avoid storing or using the sensor where it may be exposed to direct sunlight.
-
- The CAL Switch function allows the user to select the display mode optimum to the displayed image.
See [“Selecting the display mode \(CAL Switch mode\)” \(page 13\)](#).
 - Selectable DICOM (page 44) Part 14 complied screen.
 - The quality control software “RadiCS LE” used to calibrate the monitor and to manage the history is included.
 - The software “ScreenManager Pro for Medical” to adjust the screen using the mouse and keyboard is included
See [“1-2. EIZO LCD Utility Disk” \(page 8\)](#).
 - Frame Synchronous mode supported (59 to 61 Hz)
 - Power saving function
This product is equipped with power saving function.
 - 0 W power consumption when the main power is off
Equipped with main power switch.
When the monitor is not required, the power supply can be shut off using the main power switch
 - Presence Sensor
The sensor on the front side of the monitor detects the movement of a person. When a person moves away from the monitor, the monitor shifts to the power saving mode automatically and does not display the images on the screen.
Therefore, the function reduces the power consumption. The sensitivity and time until the power saving mode is activated can be set according to the monitor usage environment and movement of the user.
See [“5-4. Setting the Power Saving Mode when a Person Leaves the Monitor” \(page 30\)](#).
 - Long service life LED backlight LCD panel
 - Displays HDCP (High-bandwidth Digital Content Protection) protected contents (DisplayPort only).

1-2. EIZO LCD Utility Disk

An “EIZO LCD Utility Disk” (CD-ROM) is supplied with this product. The following table shows the disk contents and the overview of the software programs.

● Disk contents and software overview

The disk includes application software programs for adjustment, and User’s Manual. Refer to Readme.txt file on the disk for software startup procedures or file access procedures.

Contents	Overview
Readme.txt file	
RadiCS LE (for Windows)	Quality control software for calibrating the monitor and managing the calibration history.
ScreenManager Pro for Medical (for Windows)	Software for adjusting the screen using the mouse and keyboard.
User’s Manual of this monitor (PDF file)	

● Using RadiCS LE/ScreenManager Pro for Medical

For the installation and use of “RadiCS LE/ScreenManager Pro for Medical”, refer to the respective User’s Manual on the disk.


When using this software, you will need to connect a PC to the monitor with the supplied USB cable. For more information, see [“7-3. Making Use of USB \(Universal Serial Bus\)” \(page 38\)](#).

1-3. Basic Operations and Functions

● Basic Adjustment menu operations


1. Displaying Adjustment menu

1. Press ENTER. The Adjustment menu appears.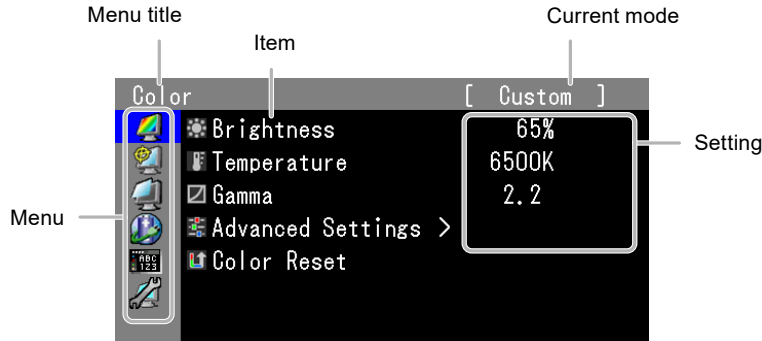


2. Adjusting/Setting


1. Select a menu to adjust/set with ▲ or ▼, and then press ENTER.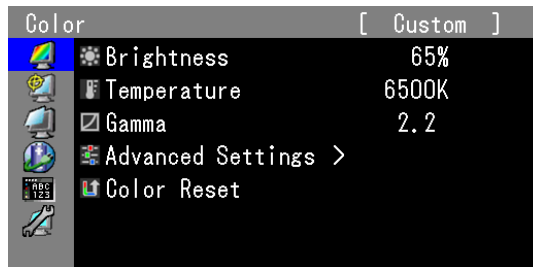

2. Select an item to adjust/set with ▲ or ▼, and then press ENTER.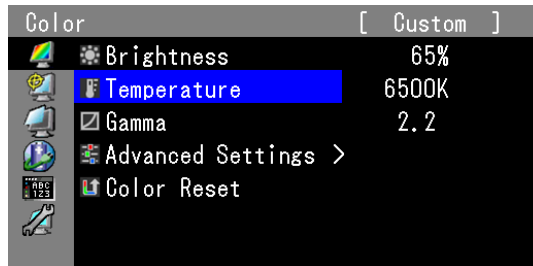

3. Adjust/set the selected item with ▲ or ▼, and then press ENTER.

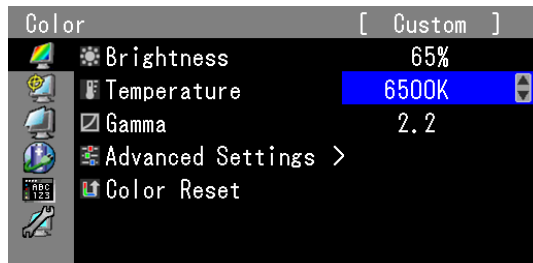3. Exiting the software

1. Press RETURN a few times. The Adjustment menu finishes.

● Functions

The following table shows the Adjustment menu and setting items of each menu.

Main Menu	Item	Reference	
Color* ¹ 	 Brightness	"2-3. Adjusting Color" (page 13)	
	 Temperature		
	 Gamma		
	 Advanced Settings  Hue  Saturation  Gain		
	 Color Reset	"3-7. Resetting to Default" (page 24)	
RadiCS SelfQC 	 SelfCalibration	"4-1. Executing Calibration" (page 25)	
	 Grayscale Check	"4-2. Checking Grayscale" (page 26)	
	 LEA	"4-3. Acquiring Life Prediction Data" (page 27)	
	 Settings	"4-4. Setting On/Off for Warning and QC History" (page 27)	
Screen 	 Screen Size	"2-5. Selecting Screen Size" (page 21)	
	 Analog Adjustment* ²  Auto Adjustment  Range Adjustment  Clock  Phase  Hor. Position  Ver. Position	"2-4. Adjusting Screens" (page 17)	
	Power Manager 	 Power Save	"5-1. Setting Power Save" (page 28)
	 Indicator	"5-2. Adjusting the Brightness of Indicator" (page 28)	
	 Presence Sensor	"5-4. Setting the Power Saving Mode when a Person Leaves the Monitor" (page 30)	
	Menu Settings 	 Language	"3-3. Setting Language" (page 23)
		 Menu Position	"3-4. Setting Adjustment Menu Display" (page 23)
Tools 	 Input Selection	"Setting input signal switching procedure" (page 37)	
	 Mode Preset	"3-2. Enabling/Disabling Mode Selection" (page 22)	
	 Signal Info	"7-4. Showing Monitor Information" (page 39)	
	 Monitor Info		
	 All Reset	"3-7. Resetting to Default" (page 24)	

Main Menu	Item	Reference
Optional Settings	Logo	“3-5. Showing and hiding the EIZO logo” (page 23)
	Key Lock	“3-6. Locking Operation Buttons” (page 24)
	Forced Analog	“4. No picture when using analog signal input” (page 32)
	DP Power Save	“5-3. Setting DisplayPort Power Save” (page 29)
	Orientation	“3-1. Setting Orientation” (page 22)

*1 The adjustment/settings available on the “Color” menu depend on the selected mode (see [“2-3. Adjusting Color” \(page 13\)](#)).

*2 These functions can be adjusted with analog input.

Chapter 2 Adjusting Screens

2-1. Compatible Resolutions

For details on compatible resolutions, refer to “Compatible Resolutions” in the Setup Manual.

2-2. Setting the Resolution

● Windows 8 / Windows 7

1. For Windows 8, click the “Desktop” tile on the Start Screen to display the desktop.
2. Right-click the mouse anywhere on the desktop except for icons.
3. From the displayed menu, click “Screen resolution”.
4. On the “Screen Resolution” dialog box, select the monitor.
5. Click “Resolution” to select the desired resolution.
6. Click the “OK” button.
7. When a confirmation dialog box is displayed, click “Keep changes”.

● Windows Vista

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Personalize”.
3. On the “Personalization” window, click “Display Settings”.
4. On the “Display Settings” dialog box, select the “Monitor” tab and select desired resolution in the “Resolution” field.
5. Click the “OK” button.
6. When a confirmation dialog box is displayed, click “Yes”.

● Windows XP

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Properties”.
3. When the “Display Properties” dialog box is displayed, click the “Settings” tab and select desired resolution for “Screen resolution” under “Display”.
4. Click the “OK” button to close the dialog box.

● Mac OS X

1. Select “System Preferences” from the Apple menu.
2. When the “System Preferences” dialog box is displayed, click “Displays” for “Hardware”.
3. On the displayed dialog box, select the “Display” tab and select desired resolution in the “Resolutions” field.
4. Your selection will be reflected immediately. When you are satisfied with the selected resolution, close the window.

2-3. Adjusting Color


● Selecting the display mode (CAL Switch mode)

A desired mode can be selected easily according to the monitor application.

Display Modes

√: Calibration is available.

Mode	Purpose	
1-DICOM	Available to display in DICOM mode.	√
2-Custom	Available for the setting according to your preference.	√
3-CAL	Displays the screen adjusted by calibration software.	√
4-Text	Suitable for displaying texts for word processing or spreadsheets.	-

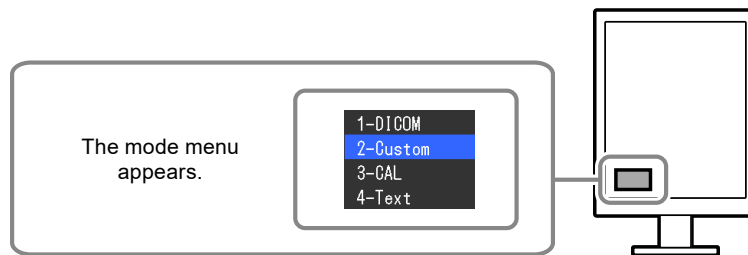Procedure

1. Press MODE.
The mode menu appears at the bottom left of the screen.
2. The mode among the list is highlighted in turn each time MODE is pressed.
You can switch the mode with ▲ or ▼ while the mode menu is displayed.

→1-DICOM←→2-Custom←→3-CAL←→4-Text←

Note

- The Adjustment menu and the Mode menu cannot be displayed at the same time.

● Performing advanced adjustments

“Color” of the Adjustment menu allows you to configure and save a separate set of color settings for each mode.

Attention

- It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.
- The same image may be seen in different colors on multiple monitors due to their monitor-specific characteristics. Make fine color adjustment visually when matching colors on multiple monitors.

Note

- The values shown in “%” or “K” are available only as reference.

● Adjustable settings in each mode

The adjustable settings depend on the mode. (Non-adjustable or non-settable function cannot be selected.)

√: Adjustable -: Not adjustable

Icon	Function	CAL Switch mode			
		1-DICOM*1	2-Custom*1	3-CAL*1	4-Text
	Brightness	√	√	√	√
	Temperature	-	√	-	√
	Gamma	-	√	-	√
	Hue	-	√	-	√
	Saturation	-	√	-	√
	Gain	-	√	-	-
	Color Reset	√	√	√	√

*1 If you adjust the calibration in this mode using the calibration kit exclusive for to this monitor (see RadiCS LE / "Accessories" (page 42)), only "Brightness" and "Color Reset" are adjustable.

● Adjusting Brightness

The screen brightness is adjusted by changing the brightness of the backlight (light source from the LCD back panel).

Setting Range

0 to 100%

Procedure

1. Select "Color" from the Adjustment menu, and press ENTER.
2. Select "Brightness" from "Color", and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

● Adjusting Temperature

The color temperature can be adjusted.

The color temperature is normally used to express the hue of "white" and/or "black" by a numerical value. The value is expressed in degrees "K" (Kelvin).

The screen becomes reddish at a low color temperature, and bluish at a high color temperature, like the flame temperature. The gain preset values are set for each color temperature setting value.

Setting Range

Native, 6000K to 15000K (specified every 100K)

Procedure

1. Select "Color" from the Adjustment menu, and press ENTER.
2. Select "Temperature" from "Color", and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

Note

- "Gain" allows you to perform more advanced adjustment (see "Adjusting Gain" (page 16)).
- If you set "Native", the image is displayed in the preset color of the monitor (Gain: 100% for each RGB).
- When gain is changed, the color temperature is changed to "User".

● Adjusting Gamma

Gamma can be adjusted. The brightness of the monitor varies depending on the input signal, however, the variation rate is not proportional to the input signal. To keep the balance between the input signal and the brightness of the monitor is called as “Gamma correction”.

Setting Range

1.6 to 2.7

Procedure

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Gamma” from “Color”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

● Adjusting Hue

The hue can be adjusted.

Setting Range

-100 to 100

Procedure

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Advanced Settings” from “Color”, and press ENTER.
3. Select “Hue”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

Attention

- After this adjustment, some color gradations may not be displayed.
-

● Adjusting Saturation

The color saturation can be adjusted.

Setting Range

-100 to 100

Procedure

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Advanced Settings” from “Color”, and press ENTER.
3. Select “Saturation”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

Attention

- After this adjustment, some color gradations may not be displayed.
-

Note

- The minimum value (-100) changes the screen to monochrome.
-

● Adjusting Gain

The brightness of each color component red, green, and blue is called Gain. You can change the hue of “white” by adjusting gain.

Setting Range

0 to 100%

Procedure

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Advanced Settings” from “Color”, and press ENTER.
3. Select “Gain”, and press ENTER.
4. Select the color for adjustment from “Red”, “Green”, and “Blue”, and press ENTER.
5. Use ▲ or ▼ for adjustment.
6. Press ENTER to exit.

Attention

- After this adjustment, some color gradations may not be displayed.

Note

- The value of gain changes with that of color temperature.
 - When gain is changed, the color temperature is changed to “User”.
-

2-4. Adjusting Screens

● Digital Signal Input

When digital signals are input, images are displayed correctly based on the preset data of the monitor, but if you perform more advanced adjustment, see “2-3. Adjusting Color” (page 13) and subsequent pages.

● Analog Signal Input

Attention

- It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.
 - The Self Adjust (automatic screen adjustment) function does not work for the images under the resolution of 800 × 600 (SVGA).
 - The self adjustment and auto adjustment functions work correctly when an image is fully displayed over the Windows or Macintosh display area. They do not work properly in the cases below:
 - when an image is displayed only on a part of the screen (command prompt window, for example)
 - when a black background (wallpaper, etc.) is in use
 - Also, these functions cannot work properly in some graphic boards.
-

The monitor screen adjustment is used to suppress flickering of the screen or adjust screen position and screen.

Note

- The self adjustment function works when all of the following conditions are satisfied:
 - When a signal is input into the monitor for the first time or when a resolution or vertical/horizontal scan frequency not used before is
-

If the screen is not displayed correctly even after performing the self adjustment operation, adjust the screen according to the procedures on the following pages to use the monitor comfortably.

How to Set

1. Perform the auto adjustment.

- To adjust flickering, screen position, and screen size automatically

Procedure

1. Choose “Screen” from the Adjustment menu, and press ENTER.
2. Choose “Analog Adjustment” from “Screen”, and press ENTER.
3. Choose “Auto Adjustment” from “Analog Adjustment”, and press ENTER.

Flickering, screen position, and screen size are corrected by the auto-adjustment function.

If the screen is not displayed correctly even after performing Auto Adjustment, perform the adjustments according to the procedures on the following pages. If the screen is displayed correctly, go to “5. Adjusting color gradation.” (page 20).

2. Prepare to use analog screen adjustment patterns.

Download the “Screen adjustment pattern files” from our site: <http://www.eizoglobal.com>

Note


- For details and instructions on opening the “Screen adjustment pattern files”, refer to the Readme.txt file.

3. Perform the auto adjustment again with the analog screen adjustment pattern displayed.

- To adjust flickering, screen position, and screen size automatically

Procedure

1. Display Pattern 1 in full screen on the monitor using the “Screen adjustment pattern files”.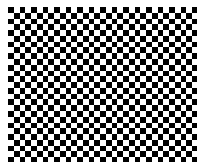

2. Choose “Screen” from the Adjustment menu, and press ENTER.
3. Choose “Analog Adjustment” from “Screen”, and press ENTER.
4. Choose “Auto Adjustment” from “Analog Adjustment”, and press ENTER.

Flickering, screen position, and screen size are corrected by the auto-adjustment function.

If the screen is not displayed correctly even after performing Auto Adjustment, perform the adjustments according to the procedures on the following pages. If the screen is displayed correctly, go to “5. Adjusting color gradation.” (page 20).

4. Choose “Screen” from the Adjustment menu to perform advanced adjustments.


Adjust the clock, phase and position, in this order.

● To eliminate vertical bars

Procedure

1. Choose “Analog Adjustment” from “Screen”, and press ENTER.
2. Choose “Clock” from “Analog Adjustment”, and press ENTER.
The “Clock” menu appears.
3. Adjust the clock with ▲ or ▼ to eliminate vertical bars.
Press ▲ or ▼ slowly so as not to miss the adjustment point.
4. Press ENTER to exit.

When blurring, flickering or bars appear on the screen after adjustment, proceed to “To remove flickering or blurring”.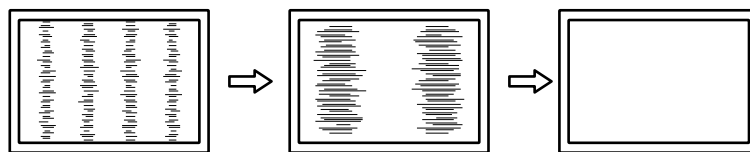


● To remove flickering or blurring

Procedure

1. Choose “Analog Adjustment” from “Screen”, and press ENTER.
2. Choose “Phase” from “Analog Adjustment”, and press ENTER.
The “Phase” menu appears.
3. Adjust the phase with ▲ or ▼ to minimize flickering or blurring.
4. Press ENTER to exit.

When vertical bars appear on the screen after adjustment, go back to “To eliminate vertical bars”.
(Clock → Phase → Position)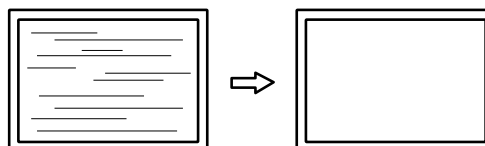

Attention

- Flickering or blurring may not be eliminated depending on your PC or graphics board.
-


● To correct the screen position

Note

- Since the number of pixels and the pixel positions are fixed on the LCD monitor, only one position is provided to display images correctly. The position adjustment is made to shift an image to the correct position.
-

Procedure

1. Choose “Analog Adjustment” from “Screen”, and press ENTER.
2. Choose “Hor. Position” or “Ver. Position” from “Analog Adjustment”, and press ENTER.
The “Hor.Position” or “Ver.Position” menu appears.
3. Adjust the position with ▲ or ▼.
4. Press ENTER to exit.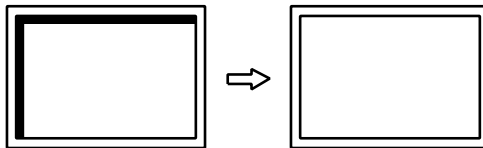

5. Close Pattern 1.


5. Adjusting color gradation.

● To adjust color gradation automatically

Every color gradation (0 to 255) can be displayed by adjusting the signal output level.

Procedure


1. Display Pattern 2 in full screen on the monitor using the “Screen adjustment pattern files”.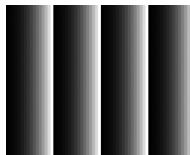

2. Choose “Screen” from the Adjustment menu, and press ENTER.
3. Choose “Analog Adjustment” from “Screen”, and press ENTER.
4. Choose “Range Adjustment” from “Analog Adjustment”, and press ENTER.
The output range is set automatically.
5. Close Pattern 2.

2-5. Selecting Screen Size

The image with the resolution other than the recommended resolution is displayed in full screen automatically. You can change the screen size by using the “Screen Size” function from “Screen”.
Example: Image size 1024 × 768 (Graphics board setting: Portrait)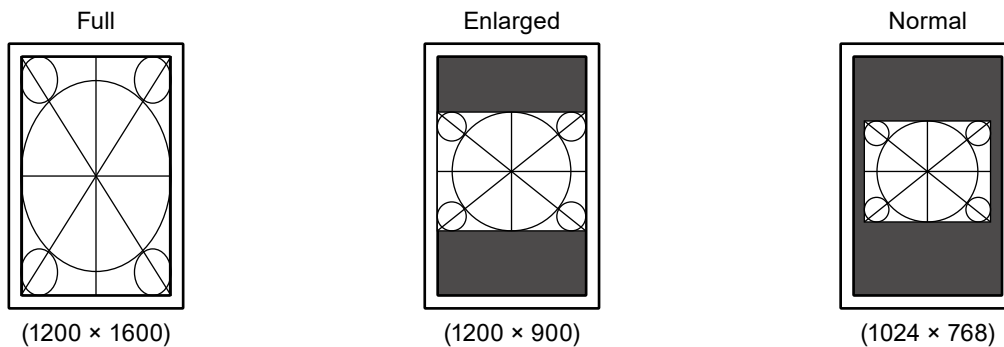

Procedure

1. Select “Screen” from the Adjustment menu, and press ENTER.
2. Select “Screen Size” from “Screen”, and press ENTER.
3. Select “Full”, “Enlarged”, or “Normal” with ▲ or ▼.

Setting	Function
Full	Displays an image in full screen. Images are distorted in some cases because the vertical rate is not equal to the horizontal rate.
Enlarged	Displays an image in full screen. In some cases, a blank horizontal or vertical border appears to equalize the vertical rate and the horizontal rate.
Normal	Displays images with the specified resolution.

4. Press ENTER to exit.

Chapter 3 Setting Monitor

3-1. Setting Orientation


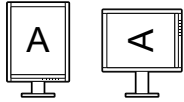
This function changes the display orientation of the image based on the operating environment.

Attention

- Refer to the User's Manual of the graphics board.
- Setting is available even when the PC is not started up.

Procedure

1. Press O to turn off the monitor.
2. Holding MODE down, press O for at least two seconds to turn on the monitor.
The "Optional Settings" menu appears.
3. Choose "Orientation" from "Optional Settings", and press ENTER.
4. Choose "Landscape", "Portrait (SW)", or "Portrait (HW)" with \blacktriangle or \blacktriangledown , and press ENTER.

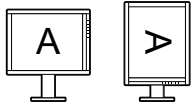Monitor Orientation	Display Example
Landscape Select this option when using the Landscape orientation.	
Portrait (SW) Select this option when using the Portrait orientation. Graphics board utility software is used to rotate the display image 90 degrees.	
Portrait (HW) Select this option when using the Portrait orientation. Graphics board utility software is used to rotate the display image 90 degrees.	

5. Select "Finish" with \blacktriangle or \blacktriangledown .
6. Press ENTER.
The "Optional Settings" menu is closed.

3-2. Enabling/Disabling Mode Selection

Only the modes currently being set can be selected.

This setting is suitable when you are using limited display modes and you do not want to change the customized display state.

Procedure

1. Select "Tools" from the Adjustment menu, and press ENTER.
2. Select "Mode Preset" from "Tools", and press ENTER.
3. Use \blacktriangle or \blacktriangledown to select the mode to change, and press ENTER.
4. Select "On" or "Off" with \blacktriangle or \blacktriangledown .
5. Press ENTER to exit.

Attention

- You cannot set all the modes disable. Set at least one mode to "On".

3-3. Setting Language

The language for the Setting menu and messages can be selected.

Selectable languages

English, German, French, Spanish, Italian, Swedish, Japanese, Simplified Chinese, Traditional Chinese

Procedure

1. Select "Menu Settings" from the Adjustment menu, and press ENTER.
2. Select "Language" from "Menu Settings", and press ENTER.
3. Select a language with ▲ or ▼.
4. Press ENTER to exit.

3-4. Setting Adjustment Menu Display

You can change the display position of the Adjustment menu.

Procedure

1. Select "Menu Settings" from the Adjustment menu, and press ENTER.
2. Select "Menu Position" from "Menu Settings", and press ENTER.
3. Select the position with ▲ or ▼.
4. Press ENTER to exit.

3-5. Showing and hiding the EIZO logo

You can choose to show or hide the display of the EIZO logo.

Procedure

1. Press ⏻ to turn off the monitor.
2. Hold down MODE while pressing ⏻ for at least two seconds to turn on the monitor.
The "Optional Settings" menu appears.
3. Select "Logo" from "Optional Settings", and press ENTER.
4. Select "On" or "Off" with ▲ or ▼, and press ENTER.
5. Select "Finish" with ▲ or ▼.
6. Press ENTER.
The "Optional Settings" menu is closed.

3-6. Locking Operation Buttons

The settings can be locked to prevent them being changed.

Procedure

1. Press O to turn off the monitor.
2. Hold down MODE while pressing O for at least two seconds to turn on the monitor.
The “Optional Settings” menu appears.
3. Select “Key Lock” from “Optional Settings”, and press ENTER.
4. Select “Off”, “Menu”, or “All” with \blacktriangle or \blacktriangledown , and press ENTER.

Item	Buttons that can be locked
Off (Default setting)	None (All buttons are enabled)
Menu	ENTER button
All	All buttons excluding O

5. Select “Finish” with \blacktriangle or \blacktriangledown .
6. Press ENTER.
The “Optional Settings” menu is closed.

3-7. Resetting to Default

There are two different Resets: Color Reset for only resetting the color adjustments to the default settings and All Reset for resetting all settings to the defaults.

Attention

- After resetting, you cannot undo the operation.

Note

- Regarding the default setting, see [“Main Default Settings” \(page 41\)](#).

● Resetting the color adjustments

Only the color adjustment value for the currently selected mode can be reset to the default settings.

Procedure

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Color Reset” from “Color”, and press ENTER.
3. Select “Execute” with \blacktriangle or \blacktriangledown .
4. Press ENTER.

The color adjustments revert to the default settings.

● Resetting all adjustments to the default settings

All settings are reset to the default settings (except the “Optional Settings” menus).

Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “All Reset” from “Tools”, and press ENTER.
3. Select “Execute” with \blacktriangle or \blacktriangledown .
4. Press ENTER.

All settings excluding the “Optional Settings” menus are reset to the default settings.

Chapter 4 Controlling Monitor Quality

This product has the built-in Integrated Front Sensor, which allows the user to perform periodic calibration and Grayscale Check for the monitor independently.

Attention

- When a panel protector is attached, the Integrated Front Sensor cannot be used. Do not execute “SelfCalibration” or “Grayscale Check” of “RadiCS SelfQC”. Be sure to set “Meas. Frequency” to “Off”.

Note

- To perform advanced quality control that conforms to medical standards, use the optional monitor quality control software kit “RadiCS UX1”.
 - To align the measurement result of the integrated front sensor with that of the UX1 sensor to be set as a reference, execute correlation using RadiCS/RadiCS LE. Periodical correlation enables the measurement accuracy of the integrated front sensor equivalent to that of the UX1 sensor. For details, refer to RadiCS/RadiCS LE User’s Manual.
 - The adjustment target for the calibration, judgment value for the grayscale check, and schedule for both of them can be set using the “RadiCS” software.
 - The adjustment target and schedule for the calibration can be set using the supplied “RadiCS LE” software. For details, refer to RadiCS LE User’s Manual (on the CD-ROM).
 - After changing the orientation of the monitor, be sure to perform correlation on the Integrated Front Sensor.
-

4-1. Executing Calibration

SelfCalibration can be executed and latest result can be viewed. In addition, when the schedule is set using RadiCS/RadiCS LE, the next calibration schedule appears.

● Executing SelfCalibration

Attention

- Do not execute this function when a panel protector is attached.
 - After SelfCalibration is executed, execute Grayscale Check under the actual temperature of the monitor usage environment.
-

Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “SelfCalibration” from “RadiCS SelfQC”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER.
SelfCalibration is executed.

● Viewing SelfCalibration result

Attention

- The result can be viewed when “QC History” is “On” (see “4-4. Setting On/Off for Warning and QC History” (page 27)).
-

Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “SelfCalibration” from “RadiCS SelfQC”, and press ENTER.
3. Select “Result” with ▲ or ▼.
4. Select “DICOM”, “Custom” or “CAL” with ▲ or ▼.
5. Press ENTER to exit.
The SelfCalibration result (Max. Error Rate, Last Time, and Error Code) appears.

4-2. Checking Grayscale

Grayscale Check can be executed and the latest result can be viewed. In addition, when the schedule is set using RadiCS, the next grayscale check schedule appears.

● Executing Grayscale Check

Attention

- Do not execute this function when a panel protector is attached.
 - Execute Grayscale Check under the actual temperature of the monitor usage environment.
-

Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Grayscale Check” from “RadiCS SelfQC”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER.
Grayscale Check is executed.

● Viewing grayscale check results

Attention

- The results can be viewed when “QC History” is “On”.
-

Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Grayscale Check” from “RadiCS SelfQC”, and press ENTER.
3. Select “Result” with ▲ or ▼.
4. Select “DICOM”, “Custom” or “CAL” with ▲ or ▼.
5. Press ENTER to exit.
Up to the five latest grayscale check results (Passed/Failed/Canceled/ERR) appear.

4-3. Acquiring Life Prediction Data

The data necessary to predict the monitor life can be acquired.

Attention

- When a panel protector is attached, be sure to set “Meas. Frequency” to “Off”.
-

Note

- Data are collected every 100 hours. Upon reaching the timing, the sensor will activate and start measurement.
 - When you select “Power Save”, data are collected when the mode switches to Power Save or the secondary power off after the above timing is reached.
 - When you select “Routine”, data are saved after the following times have elapsed.
 - - 500, 1000, 2000, 4000, 7000, 10000, 15000, 20000, 25000, 30000 hours
-

Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “LEA” from “RadiCS SelfQC”, and press ENTER.
3. Select “Meas. Frequency” with ▲ or ▼.
4. Select “Power Save”, “Routine”, or “Off” with ▲ or ▼.
5. Press ENTER to exit.

4-4. Setting On/Off for Warning and QC History

The warning and QC history displays for the failure of the Grayscale Check result can be set to on/off.

Procedure

1. Select “RadiCS SelfQC” from the Adjustment menu, and press ENTER.
2. Select “Settings” from “RadiCS SelfQC”, and press ENTER.
3. Select “Warning” or “QC History” with ▲ or ▼.
4. Select “On” or “Off” with ▲ or ▼.
5. Press ENTER to exit.

Chapter 5 Power Saving Functions

5-1. Setting Power Save

The monitor can be set to the power saving mode according to the PC status. When the monitor has shifted to the power saving mode, the images on the screen are not displayed.

Attention

- Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.
- Devices connected to the USB downstream port work even when the monitor is in power saving mode or when the power button of the monitor is Off. Therefore, power consumption of the monitor varies with connected devices even in the power saving mode.
- Five seconds before the monitor enters the power saving mode, it displays a preliminary message.

Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Power Save” from “PowerManager”, and press ENTER.
3. Select “Off” and either “High” or “Low” with ▲ or ▼.

Setting	Function
High	Turns off the backlight. The power saving effect is the greatest.
Low	Turns down the backlight to the lowest setting. The time until the operation stabilizes after exiting the Power Save mode next time can be reduced.

4. Press ENTER to exit.

Power Saving System

At DVI signal input

This monitor complies with the “DVI DMPM” standard.

At DisplayPort signal input

This monitor complies with the “DisplayPort Standard V1.1a”.

The monitor enters the power saving mode after five seconds in connection with the PC setting.

PC	Monitor	Power indicator
On	Operation mode	Green
Power saving mode	Power saving mode	Orange

5-2. Adjusting the Brightness of Indicator

The brightness of the power indicator (green) when the screen is displayed can be set. (The power indicator is set to light up when the power is switched on, and set to 4 by default.)

Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Indicator” from “PowerManager”, and press ENTER.
3. Select the indicator brightness “Off” or in the range from 1 to 7 with ▲ or ▼ as desired.
4. Press ENTER to exit.

5-3. Setting DisplayPort Power Save

The power setting of the DisplayPort of the monitor can be switched in the power saving mode.

Attention

- When “On” is set, the display position on the window may not be positioned correctly when the monitor power is turned on or off. In such a case, turn the setting to “Off”.
-

Procedure

1. Press \odot to turn off the monitor.
2. Hold down MODE while pressing \odot for at least two seconds to turn on the monitor.
The “Optional Settings” menu appears.
3. Select “DP Power Save” from “Optional Settings”, and press ENTER.
4. Select “On” or “Off” with \blacktriangle or \blacktriangledown , and press ENTER.
5. Select “Finish” with \blacktriangle or \blacktriangledown .
6. Press ENTER.
The “Optional Settings” menu is closed.

5-4. Setting the Power Saving Mode when a Person Leaves the Monitor

The sensor on the front of the monitor detects the movement of the operator when Presence Sensor is set to “On”. When a person moves away from the monitor, it shifts to the power saving mode automatically and does not display the images on the screen to reduce the power consumption. When a person comes near the monitor again, it recovers from the power saving mode and displays the images. The sensitivity and time until the power saving mode is activated can be set according to the monitor usage environment and movement of the user.

Note

- At the time of shifting to power saving mode, the message that indicates shifting is displayed.

Attention

- This setting cannot be used during “RadiCS SelfQC” execution.

Procedure

1. Select “PowerManager” from the Adjustment menu, and press ENTER.
2. Select “Presence Sensor” from “PowerManager”, and press ENTER.
3. Select “On” or “Off” with ▲ or ▼.
4. Press ENTER to exit.
5. After selecting “On”, set the details.

Item	Setting Range	Description
Time	5, 30 sec 1, 3, 5, 10, 15, 30, 45, 60 min	Specifies the time from when a person leaves the monitor until the “No Presence detected” message appears. The monitor shifts to the power saving mode about 20 seconds after the message is displayed.
Sensitivity	Level 5 Level 4 Level 3 Level 2 Level 1	When the setting level is high (“Level 5” at maximum), even minor movements of a person are detected, causing the monitor to shift to the power saving mode only infrequently. On the other hand, when the setting level is low (“Level 1” at minimum), only major movements of a person are detected, allowing the monitor to shift to the power saving mode frequently. The Presence Sensor function is activated when the sensor on the front of the monitor detects the movement of heat. Therefore, the sensitivity should be changed depending on the typical user’s clothes or the typical room temperature. If the monitor shifts to or recovers from the power saving mode contrary to your expectations, try changing the setting level.
Reset	-	This function resets all adjustments to the default settings.

Note

- To use the monitor comfortably, increase the sensitivity for high temperatures, which reduces the detection sensitivity, and reduce the sensitivity for low temperatures, which increases the detection sensitivity.

Chapter 6 Troubleshooting

If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

6-1. No Picture

Problem	Possible cause and remedy
<p>1. No picture</p> <ul style="list-style-type: none"> Power indicator does not light. <hr/> <ul style="list-style-type: none"> Power indicator is lighting green. <hr/> <ul style="list-style-type: none"> Power indicator is lighting orange. <hr/> <ul style="list-style-type: none"> Power indicator is flashing orange and green. 	<ul style="list-style-type: none"> Check whether the power cord is connected properly. Turn the main power switch on. Press . Turn off the main power, and then turn it on again a few minutes later. <hr/> <ul style="list-style-type: none"> Increase "Brightness" and/or "Gain" in the Adjustment menu (see "Performing advanced adjustments" (page 13)). <hr/> <ul style="list-style-type: none"> Switch the input signal with SIGNAL. Move the mouse or press any key on the keyboard. Check whether the PC is turned on. When Presence Sensor is set to "On", the monitor may have shifted to power saving mode. Come near to the monitor. <hr/> <ul style="list-style-type: none"> The device that is connected using DisplayPort have a problem. Solve the problem, turn off the monitor, and then turn it on again. Refer to the User's Manual of the output device for further details.
<p>2. The message below appears.</p> <ul style="list-style-type: none"> This message appears when no signal is input. Example: <div data-bbox="269 1220 673 1480" data-label="Image"> </div> <ul style="list-style-type: none"> The message shows that the input signal is out of the specified frequency range. (Such signal frequency is displayed in magenta.) Example: <div data-bbox="269 1628 673 1973" data-label="Image"> </div> <p>fD: Dot clock fH: Horizontal scan frequency fV: Vertical scan frequency</p>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <ul style="list-style-type: none"> The message shown left may appear, because some PCs do not output the signal soon after power-on. Check whether the PC is turned on. Check whether the signal cable is connected properly. Switch the input signal with SIGNAL. <hr/> <ul style="list-style-type: none"> Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the Setup Manual.). Reboot the PC. Select an appropriate setting using the graphics board's utility. Refer to the manual of the graphics board for details.

6-2. Imaging Problems (for both digital and analog)

Problem	Possible cause and remedy
1. The screen is too bright or too dark.	<ul style="list-style-type: none"> Use "Brightness" in the Adjustment menu to adjust it. (The LCD monitor backlight has a limited life span. When the screen becomes dark or begins to flicker, contact your local EIZO representative.)
2. Characters are blurred.	<ul style="list-style-type: none"> Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the Setup Manual.).
3. Afterimages appear.	<ul style="list-style-type: none"> Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time. Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.
4. Green/red/blue/white dots or defective dots remain on the screen.	<ul style="list-style-type: none"> This is due to LCD panel characteristics and is not a failure.
5. Interference patterns or pressure marks remain on the screen.	<ul style="list-style-type: none"> Leave the monitor with a white or black screen. The symptom may disappear.

6-3. Imaging Problems (for analog only)

Problem	Possible cause and remedy
1. Display position is incorrect. 	<ul style="list-style-type: none"> Use "Position" in the Adjustment menu to correct the image position (see "To correct the screen position" (page 20)). If the problem persists, use the graphics board's utility if available to change the display position.
2. Vertical bars appear on the screen or a part of the image is flickering. 	<ul style="list-style-type: none"> Use "Clock" in the Adjustment menu to adjust it (see "To eliminate vertical bars" (page 19)).
3. Whole screen is flickering or blurring. 	<ul style="list-style-type: none"> Use "Phase" in the Adjustment menu to adjust it (see "To remove flickering or blurring" (page 19)).
4. No picture when using analog signal input	<ul style="list-style-type: none"> Check whether a signal cable specified by us is being used. Execute the following procedure. <ol style="list-style-type: none"> Press  to turn off the monitor. Hold down MODE while pressing  for at least two seconds to turn on the monitor. The "Optional Settings" menu appears. Select "Forced Analog" from "Optional Settings", and press ENTER. Select "On" with  or  and press ENTER. This forces recognition of the input signal as analog. (Digital signals cannot be displayed.)

6-4. Other Problems

Problem	Possible cause and remedy
1. The adjustment menu does not appear.	<ul style="list-style-type: none"> • Check whether the operation lock function is active (see “3-6. Locking Operation Buttons” (page 24)).
2. No mode menu is displayed.	<ul style="list-style-type: none"> • Check whether the operation lock function is active (see “3-6. Locking Operation Buttons” (page 24)).
3. (When using the presence sensor) The images remain to be displayed when you move away from the monitor. / The images do not reappear when you come near it.	<ul style="list-style-type: none"> • Check the setting environment of the monitor. Presence Sensor may not work correctly in the following environments. <ul style="list-style-type: none"> - The monitor is located in a place affected by wind. - There is some heat emitting device near the monitor. - There is some obstacle in front of the sensor. • Check whether any dirt on the sensor. Clean the sensor with a soft cloth. • Check whether you are in front of the monitor. The Presence Sensor function is activated when the sensor on the front of the monitor detects the movement of heat. • In the high temperature conditions, the monitor may not recover from the power saving mode. If the screen is not displayed even after moving the mouse or pressing any key on the keyboard, turn the monitor off and on using its power button. When the screen is displayed, increase the detection sensitivity value (see “5-4. Setting the Power Saving Mode when a Person Leaves the Monitor” (page 30)).
4. The monitor connected with the USB cable is not detected. / USB devices connected to the monitor does not work.	<ul style="list-style-type: none"> • Check whether the USB cable is connected correctly (see “7-3. Making Use of USB (Universal Serial Bus)” (page 38)). • Change the USB port to another one. If the PC or peripheral devices works correctly by changing the USB port, contact your local EIZO representative. (Refer to the manual of the PC for details.) • Reboot the PC. • If the peripheral devices work correctly when the PC and peripheral devices are connected directly, please contact your local EIZO representative. • Check whether the PC and its OS are USB compliant. (For USB compliance of the respective devices, consult their manufacturers.) • For Windows, check the PC’s BIOS setting for USB. (Refer to the manual of the PC for details.)
5. Integrated Front Sensor stays out.	<ul style="list-style-type: none"> • Turn the main power off and on again.
6. SelfCalibration/Grayscale Check failure	<ul style="list-style-type: none"> • Refer to the error code table. • If an error code that does not appear in the error code table is displayed, contact your local EIZO representative.
7. Time is not displayed correctly in the history of SelfCalibration/Grayscale Check	<ul style="list-style-type: none"> • Execute Monitor Detection using RadiCS/RadiCS LE. For details, refer to RadiCS/RadiCS LE User’s Manual.

● Error code table

Error Code	Description
0***	<ul style="list-style-type: none"> • Errors that occurred during SelfCalibration
1***	<ul style="list-style-type: none"> • Errors that occurred during Grayscale Check
*1**	<ul style="list-style-type: none"> • Errors that occurred during DICOM
*2**	<ul style="list-style-type: none"> • Errors that occurred during Custom
*3**	<ul style="list-style-type: none"> • Errors that occurred during CAL
**10	<ul style="list-style-type: none"> • The product maximum brightness may be lower than the target brightness. • Lower the target brightness.
**11	<ul style="list-style-type: none"> • The product minimum brightness may be higher than the target brightness. • Raise the target brightness.
**34	<ul style="list-style-type: none"> • The sensor may not have come out during calibration, or light may have entered the sensor. • Turn off the main power, wait for a few minutes before turning the power back on, and then execute SelfCalibration/Grayscale Check again.
**61	<ul style="list-style-type: none"> • The sensor may not have come out. • Check whether there is any foreign object near the sensor. • Execute SelfCalibration/Grayscale Check again.
**95	<ul style="list-style-type: none"> • The execution conditions for SelfCalibration/Grayscale Check may not be appropriate. • Check the execution conditions and reset them if necessary.

Chapter 7 Reference

7-1. Attaching Arm

An optional arm (or an optional stand) can be attached by removing the stand section. Please refer to our web site for the corresponding optional arm (or optional stand). <http://www.eizoglobal.com>

Attention

- When attaching an arm or stand, follow the instructions of their User's Manual.
- When using another manufacturer's arm or stand, confirm the following in advance and select one conforming to the VESA standard.
 - - Clearance between the screw holes: 100 mm × 100 mm
 - - Thickness of plate: 2.6 mm
 - - Strong enough to support weight of the monitor unit (except the stand) and attachments such as cables.
- When using another manufacturer's arm or stand, use the following screws to secure it.
 - - Screws fixing the stand to the monitor
- When using an arm or stand, attach it to meet the following tilt angles of the monitor.
 - - Up 45°, down 45° (vertical display, and horizontal display rotated 90 degrees counter-clockwise)
- Connect the cables after attaching an arm or stand.
- Do not move the removed stand up and down. Doing so may result in injury or device damage.
- The monitor and arm or stand are heavy. Dropping them may result in injury or equipment damage.

Procedure


1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.

2. Remove the stand.

Prepare a screwdriver. Loosen the screws (four) securing the unit and the stand with the screwdriver.

3. Attach the arm or stand to the monitor.


Secure the monitor to the arm or stand using the screws specified in the User's Manual of the arm or stand.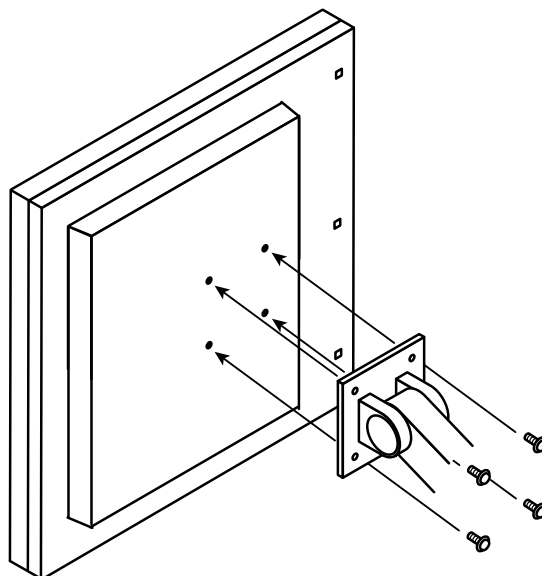

7-2. Connecting Multiple PCs

The product has multiple connections to PCs and allows you to switch to one of the connections for display.

Connection examples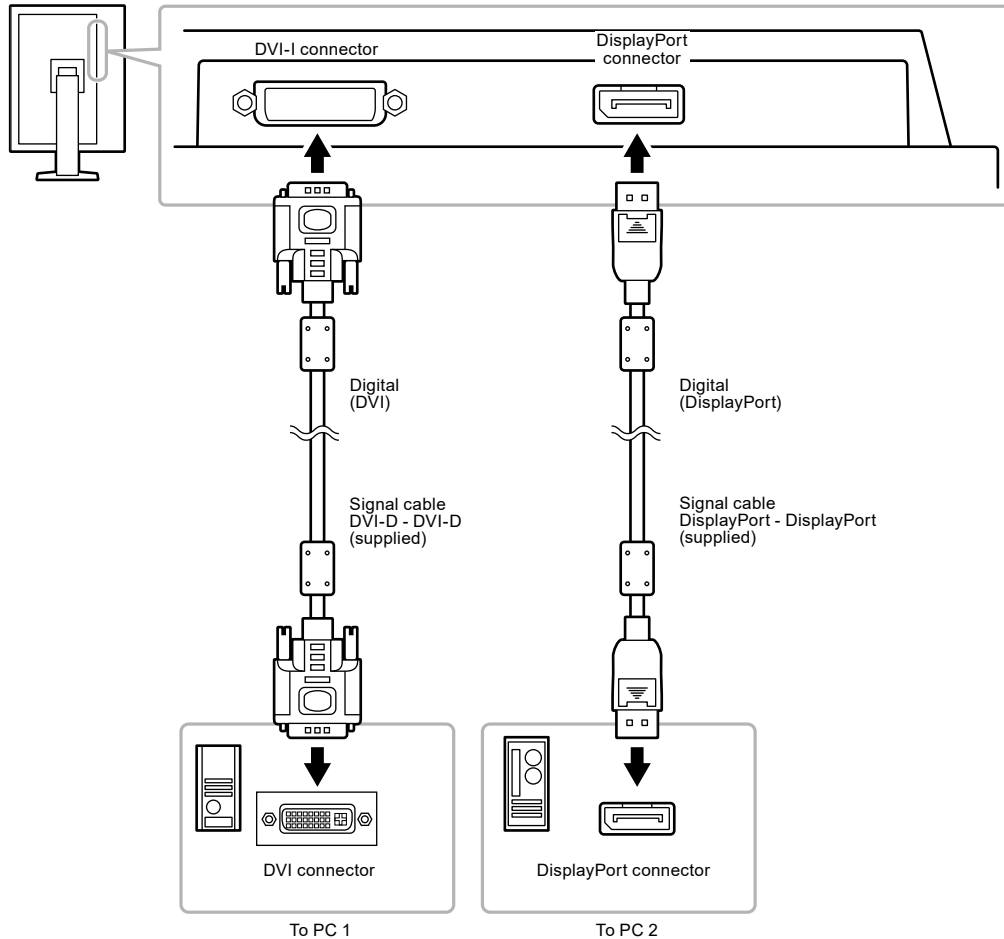


● Switching among input signals

Procedure

The input signal switches each time SIGNAL is pressed.

When the signal is switched, the active input port name appears at the top right corner of the screen.

(Example)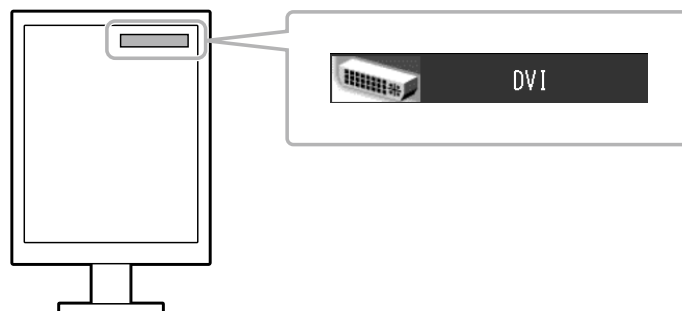

● Setting input signal switching procedure

Setting	Function
Auto	The monitor recognizes the connector through which PC signals are input. When a PC is turned off or enters the power saving mode, the monitor automatically displays another signal.
Manual	The monitor displays the signal from the selected connector despite the signal input. Select an active input signal with SIGNAL.

Procedure

1. Select "Tools" from the Adjustment menu, and press ENTER.
2. Select "Input Selection" from "Tools", and press ENTER.
3. Select "Auto" or "Manual" with ▲ or ▼.
4. Press ENTER to exit.

Note

- If you select "Auto", the monitor enters the power saving mode after all the PCs have entered the power saving mode or have been switched off.
-

7-3. Making Use of USB (Universal Serial Bus)

This monitor has a USB hub. It works as a USB hub when connected to a USB-compatible PC, allowing the connection of peripheral USB devices.

● Required system environment

- A PC equipped with a USB port
- Windows 8/7/Vista/XP/2000 or Mac OS 9.2.2 and Mac OS X 10.2 or later
- USB cable

Attention


- This monitor may not work depending on PC, OS or peripheral devices to be used. For USB compatibility of peripheral devices, contact their manufacturers.
- Devices connected to the USB downstream port work when the monitor is in power saving mode or when the power button of the monitor is Off.
- When the main power of the monitor is Off, a device connected to the USB downstream port will not operate.

● Connection procedure (Setup of USB function)

1. Connect the monitor first to a PC using the signal cable, and run the PC.
2. Connect the USB cable between the USB downstream port of the PC and the USB upstream port of the monitor.

The USB hub function is set up automatically upon connection of the USB cable.

3. Connect the peripheral USB devices to the USB downstream port of the monitor.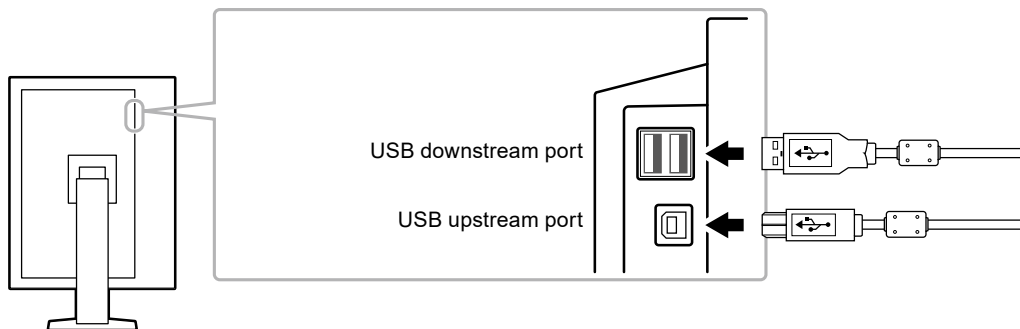

7-4. Showing Monitor Information

● Displaying signal information


The information about the current input signals can be displayed.

Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Signal Info” from “Tools”, and press ENTER.

The “Signal Info” appears.

(Example)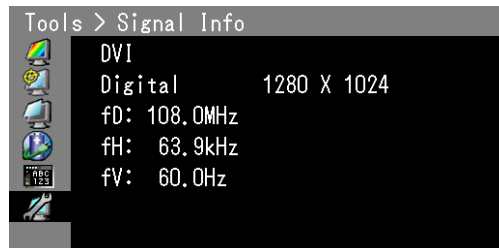

● Showing monitor information


The information about the monitor can be displayed.

Procedure

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Monitor Info” from “Tools”, and press ENTER.

The “Monitor Info” appears.

The model name, serial number, firmware version, and usage time can be displayed.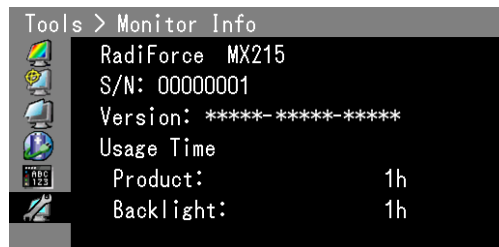

Attention

- When you turn on the monitor for the first time, the usage time may be more than 0 due to a factory inspection.
-

7-5. Specifications

LCD panel	Size	21.3 inch (540 mm)	
	Type	Color TFT, LED backlight	
	Surface treatment	Anti-glare	
	Surface hardness	3H	
	Viewing angles	Horizontal 178°, vertical 178° (CR≥10)	
	Dot pitch	0.270 mm	
	Response time	Black-white-black: Approx. 20 ms	
Horizontal scan frequency	Analog	26 - 100 kHz	
	Digital	31 - 100 kHz	
Vertical scan frequency	Analog	49.0 - 76 Hz (non-interlace) (VGA TEXT: 69 - 71 Hz)	
	Digital	59.0 - 61 Hz (non-interlace) (VGA TEXT: 69 - 71 Hz)	
Resolution		2M pixels (Portrait orientation: 1200 dots × 1600 lines (H × V))	
Max. dot clock	Analog	164.5 MHz	
	Digital	164.5 MHz	
Max. display color		Approx. 1073.74 million colors (for DisplayPort 10 bit input)	
Recommended brightness		180 cd/m ²	
Display area (H × V)		324.0 mm (12.8 inch) × 432.0 mm (17.0 inch) (Portrait orientation)	
Power supply		100 - 120 Vac ±10%, 50/60 Hz 0.5 - 0.4A 200 - 240 Vac ±10%, 50/60 Hz 0.3 - 0.25A	
Power consumption	Screen display On	48 W or less	
	Power saving mode	0.5 W or less (when only the DVI signal connector is connected, analog signal input, no USB device is connected, "Input Selection" is set to "Manual", and "DP Power Save" is set to "On")	
	Power Off	0.5 W or less (when no USB device is connected, "DP Power Save" is set to "On")	
	Main power Off	0 W	
Input signal connectors		DVI-I connector DisplayPort connector (Standard V1.1a, applicable to HDCP)	
Analog Input Signal (Sync)		Separate, TTL, positive / negative Composite, TTL, positive / negative	
Analog Input Signal (Video)		Analog, Positive (0.7 Vp-p / 75 Ω)	
Digital Signal (DVI) Transmission System		TMDS (Single Link)	
Plug & Play		Digital (DVI-I): VESA DDC 2B / EDID structure 1.3 Digital (DisplayPort): VESA DisplayPort / EDID structure 1.4	
Dimensions (width) × (height) × (depth)	Main unit	360 mm (14.2 inch) × 505 - 597.5 mm (19.9 - 23.5 inch) × 245.5 mm (9.7 inch)	
	Main unit (without stand)	360 mm (14.2 inch) × 485 mm (19.1 inch) × 64 mm (2.52 inch)	
Mass	Main unit	Approx. 8.0 kg (17.6 lbs.)	
	Main unit (without stand)	Approx. 5.4 kg (11.9lbs.)	
Movable range	FlexStand	Tilt:	Up 30°, down 0°
		Swivel:	344°
		Adjustable height:	174 mm (Tilt: 0°)
		Rotation:	90° (counterclockwise for portrait display)

Environmental conditions	Temperature	Operating: 0°C - 35°C (32°F - 95°F) Transportation/Storage: -20°C - 60°C (-4°F - 140°F)
	Humidity	Operating: 20% - 80% R.H. (no condensation) Transportation/Storage: 10% - 90% R.H. (no condensation)
	Air pressure	Operating: 700 hPa - 1060 hPa Transportation/Storage: 200 hPa - 1060 hPa
USB	Standard	USB Specification Revision 2.0
	Port	Upstream port, 1 pc., Downstream port, 2 pcs.
	Communication speed	480 Mbps (high), 12 Mbps (full), 1.5 Mbps (low)
	Supply current	Downstream: Max. 500mA/1 port

Main Default Settings

CAL Switch Mode: The default display setting is 1-DICOM.

Mode	Brightness	Temperature	Gamma value
1-DICOM	180 cd/m ²	7,500 K	DICOM
2-Custom	Approx. 370 cd/m ²	7,500 K	2.2
3-CAL	180 cd/m ²	7,500 K	DICOM
4-Text	Approx. 100 cd/m ²	6,500 K	2.2


Others

Presence Sensor	Off
Power Save	High
DP Power Save* ¹	Off
Input Selection	Auto
Logo* ¹	Off
Key Lock* ¹	Off
Orientation* ¹	Portrait(HW)
Screen Size	Normal
Menu Position	Center
Language	English

*¹ These menus cannot be initialized with "All Reset" (see [page 24](#)).

Outside Dimensions

Unit: mm

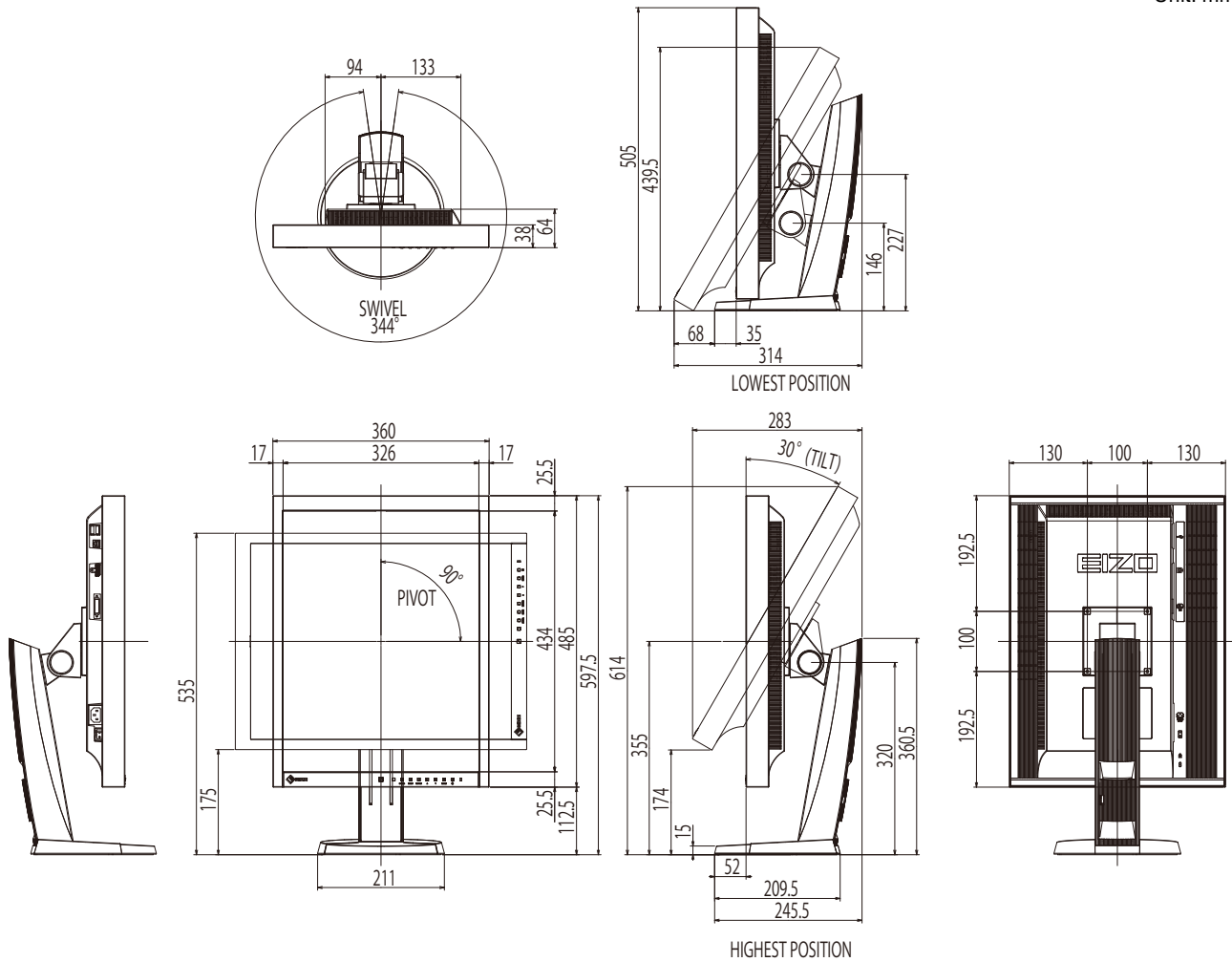Accessories

Calibration Kit	EIZO "RadiCS UX1" Ver. 4.1.4 or later EIZO "RadiCS Version Up Kit" Ver. 4.1.4 or later
Network QC Management Software	EIZO "RadiNET Pro" Ver. 4.1.4 or later
Cleaning Kit	EIZO "ScreenCleaner"
Panel protector*1	FP-2101

*1 When a panel protector is attached, the Integrated Front Sensor cannot be used.

For the latest information about the accessories and information about the latest compatible graphics board, refer to our web site.

<http://www.eizoglobal.com>

7-6. Preset Timing

The following table shows factory preset analog signal timing.

Attention

- Display position may be deviated depending on the PC connected, which may require screen adjustment using Setting menu.
- If a signal other than those listed in the table is input, adjust the screen using the Setting menu. However, screen display may still be incorrect even after the adjustment.
- When interlace signals are used, the screen cannot be displayed correctly even after screen adjustment using the Setting menu.

Resolution	Applicable Signals	Frequency			Polarity	
		Dot Clock: MHz	Horizontal: kHz	Vertical: Hz	Horizontal	Vertical
640 × 480	VGA	25.18	31.47	59.94	Negative	Negative
640 × 480	VESA	31.50	37.86	72.81	Negative	Negative
640 × 480	VESA	31.50	37.50	75.00	Negative	Negative
720 × 400	VGA TEXT	28.32	31.47	70.09	Negative	Positive
800 × 600	VESA	36.00	35.16	56.25	Positive	Positive
800 × 600	VESA	40.00	37.88	60.32	Positive	Positive
800 × 600	VESA	50.00	48.08	72.19	Positive	Positive
800 × 600	VESA	49.50	46.88	75.00	Positive	Positive
1024 × 768	VESA	65.00	48.36	60.00	Negative	Negative
1024 × 768	VESA	75.00	56.48	70.07	Negative	Negative
1024 × 768	VESA	78.75	60.02	75.03	Positive	Positive
1152 × 864	VESA	108.00	67.50	75.00	Positive	Positive
1200×1600	VESA	162.25	99.42	59.96	Negative	Positive
1280 × 960	VESA	108.00	60.00	60.00	Positive	Positive
1280 × 1024	VESA	108.00	63.98	60.02	Positive	Positive
1280 × 1024	VESA	135.00	79.98	75.03	Positive	Positive
1600 × 1200	VESA	162.00	75.00	60.00	Positive	Positive
640 × 480	Macintosh	30.24	35.00	66.67	Negative	Negative
832 × 624	Macintosh	57.28	49.72	74.55	Negative	Negative
1152 × 870	Macintosh	100.00	68.68	75.06	Negative	Negative
1280 × 960	Macintosh	126.20	74.76	74.76	Positive	Positive
640 × 400	PC-9821 AP2	25.18	31.48	70.10	Negative	Negative

Chapter 8 Glossary

Clock

The analog signal input monitor needs to reproduce a clock of the same frequency as the dot clock of the graphics system in use, when the analog signal is converted to a digital signal for image display. This is called clock adjustment. If the clock pulse is not set correctly, some vertical bars appear on the screen.

DICOM (Digital Imaging and Communication in Medicine)

The DICOM standard was developed by the American College of Radiology and the National Electrical Manufacturer's Association of the USA.

The DICOM compatible device connection enables to transfer the medical image and information. The DICOM, Part 14 document defines the digital, grayscale medical image display.

DisplayPort

DisplayPort is a next-generation digital AV interface that allows connection of the PC, audio, imaging devices, etc. to the monitor. One cable can transfer sound with images.

DVI (Digital Visual Interface)

DVI is a digital interface standard. DVI allows direct transmission of the PC's digital data without loss.

This adopts the TMDS transmission system and DVI connectors. There are two types of DVI connectors. One is a DVI-D connector for digital signal input only. The other is a DVI-I connector for both digital and analog signal inputs.

DVI DMPM (DVI Digital Monitor Power Management)

DVI DMPM is a digital interface power saving function. The "Monitor ON (operating mode)" and "Active Off (power saving mode)" are indispensable for DVI DMPM as the monitor's power mode.

Gain

This is used to adjust each color parameter for red, green and blue. An LCD monitor displays the color by the light passing through the panel color filter. Red, green and blue are the three primary colors. All the colors on the screen are displayed by combining these three colors. The color tone can be changed by adjusting the light intensity (volume) passing through each color's filter.

Gamma

Generally, the monitor brightness varies nonlinearly with the input signal level, which is called "Gamma Characteristic". A small gamma value produces a low-contrast image, while a large gamma value produces a highcontrast image.

HDCP (High-bandwidth Digital Content Protection)

Digital signal coding system developed to copy-protect the digital contents, such as video, music, etc.

This helps to transmit the digital contents safely by coding the digital contents sent via the DVI or HDMI connector on the output side and decoding them on the input side.

Any digital contents cannot be reproduced if both of the equipments on the output and input sides are not applicable to HDCP system.

Phase

Phase means the sampling timing to convert the analog input signal to a digital signal. Phase adjustment is made to adjust the timing. It is recommended that phase adjustment be made after the clock is adjusted correctly.

Range Adjustment

Range adjustment controls the signal output levels to display every color gradation. It is recommended that range adjustment be made before color adjustment.

Resolution

The LCD panel consists of numerous pixels of specified size, which are illuminated to form images. This monitor consists of horizontal 1200 pixels and 1600 vertical pixels. At a resolution of 1200 × 1600 (Portrait orientation), all pixels are illuminated as a full screen (1:1).

Temperature

Color temperature is a method to measure the white color tone, generally indicated in degrees Kelvin. The screen becomes reddish at a low temperature, and bluish at a high temperature, like the flame temperature.

5000 K: Slightly reddish white

6500 K: White referred to as daylight-balanced color

9300 K: Slightly bluish white

TMDs (Transition Minimized Differential Signaling)

A signal transmission system for digital interface.

VESA DPM (Video Electronics Standards Association - Display Power Management)

VESA specifications achieve increased energy efficiency for computer monitors. This involves standardization of signals sent from the computer (graphics board).

DPM defines the status of signals transmitted between the computer and monitor.

Appendix

Trademark

The terms HDMI and HDMI High-Definition Multimedia Interface, and the HDMI Logo are trademarks or registered trademarks of HDMI Licensing, LLC in the United States and other countries.

The DisplayPort Compliance Logo and VESA are registered trademarks of the Video Electronics Standards Association.

Acrobat, Adobe, Adobe AIR, and Photoshop are registered trademarks of Adobe Systems Incorporated in the United States and other countries.

AMD Athlon and AMD Opteron are trademarks of Advanced Micro Devices, Inc.

Apple, ColorSync, eMac, iBook, iMac, iPad, Mac, MacBook, Macintosh, Mac OS, PowerBook, and QuickTime are registered trademarks of Apple Inc.

ColorMunki, Eye-One, and X-Rite are registered trademarks or trademarks of X-Rite Incorporated in the United States and/or other countries.

ColorVision and ColorVision Spyder2 are registered trademarks of DataColor Holding AG in the United States.

Spyder3 and Spyder4 are trademarks of DataColor Holding AG.

ENERGY STAR is a registered trademark of the United States Environmental Protection Agency in the United States and other countries.

GRACoL and IDEAlliance are registered trademarks of International Digital Enterprise Alliance.

NEC is a registered trademark of NEC Corporation.

PC-9801 and PC-9821 are trademarks of NEC Corporation.

NextWindow is a registered trademark of NextWindow Ltd.

Intel, Intel Core and Pentium are registered trademarks of Intel Corporation in the United States and other countries.

PowerPC is a registered trademark of International Business Machines Corporation.

PlayStation is a registered trademark of Sony Computer Entertainment Inc.

PSP and PS3 are trademarks of Sony Computer Entertainment Inc.

RealPlayer is a registered trademark of RealNetworks, Inc.

TouchWare is a trademark of 3M Touch Systems, Inc.

Windows, Windows Media, Windows Vista, SQL Server, and Xbox 360 are registered trademarks of Microsoft Corporation in the United States and other countries.

YouTube is a registered trademark of Google Inc.

Firefox is a registered trademark of the Mozilla Foundation.

Kensington and MicroSaver are trademarks of ACCO Brands Corporation.

EIZO, the EIZO Logo, ColorEdge, DuraVision, FlexScan, FORIS, RadiCS, RadiForce, RadiNET, Raptor, and ScreenManager are registered trademarks of EIZO Corporation in Japan and other countries.

ColorNavigator, EcoView NET, EIZO EasyPIX, EIZO ScreenSlicer, i•Sound, Screen Administrator, and UniColor Pro are trademarks of EIZO Corporation.

All other company and product names are trademarks or registered trademarks of their respective owners.

License

The bitmap font used for this product is designed by Ricoh Industrial Solutions Inc.

Medical Standards

- It shall be assured that the final system is in compliance to IEC60601-1-1 requirement.
- Power supplied equipment can emit electromagnetic waves, that could influence, limit or result in malfunction of the monitor. Install the equipment in a controlled environment, where such effects are avoided.

Classification of Equipment

- Type of protection against electric shock: Class I
- EMC class: IEC60601-1-2 Group 1 Class B
- Classification of medical device (EU): Class I
- Mode of operation: Continuous
- IP Class: IPX0

EMC Information

The RadiForce series has a performance that appropriately displays medical images.

Environments of Intended Use

The RadiForce series is intended to be used in the following environments.

- Professional healthcare facility environments such as clinics and hospitals

The following environments are not suitable for the RadiForce series to be used:

- Home healthcare environments
- In the vicinity of high-frequency surgical equipments such as electrosurgical knives
- In the vicinity of short-wave therapy equipments
- RF shielded room of the medical equipment systems for MRI
- In shielded location Special environments
- Installed in vehicles including ambulances
- Other special environment

WARNING

The RadiForce series requires special precautions regarding EMC and need to be installed. You need to carefully read EMC Information and the “PRECAUTIONS” section in this document, and observe the following instructions when installing and operating the product.

The RadiForce series should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

When using a portable RF communication equipment, keep it 30 cm (12 inches) or more away from any part, including cables, of the RadiForce series. Otherwise, degradation of the performance of this equipment could result.

Anyone who connects additional equipment to the signal input part or signal output parts, configuring a medical system, is responsible that the system complies with the requirements of IEC60601-1-2.

Do not touch the signal input/output connectors while using the RadiForce series. Otherwise, the displayed image may be affected.

Be sure to use the cables attached to the product, or cables specified by EIZO.

Use of cables other than those specified or provided by EIZO of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and improper operation.

Cables	EIZO Designated Cables	Max. Cable Length	Shielding	Ferrite Core
Signal cable (DisplayPort)	PP300 / PP200	3 m	Shielded	With Ferrite Cores
Signal cable (DVI-D)	DD300 / FD-C39	3 m	Shielded	With Ferrite Cores
Signal cable (DVI-I)	FD-C16	2 m	Shielded	With Ferrite Cores
USB cable	UU300 / MD-C93	3 m	Shielded	With Ferrite Cores
Power cord (with earth)	-	3 m	Unshielded	Without Ferrite Cores

Technical Descriptions

Electromagnetic emissions		
The RadiForce series is intended for use in the electromagnetic environment specified below. The customer or the user of the RadiForce series should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment - Guidance
RF emissions CISPR11	Group 1	The RadiForce series uses RF energy only for its internal function. Therefore, its RF emission are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11	Class B	The RadiForce series is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC61000-3-2	Class D	
Voltage fluctuations / flicker emissions IEC61000-3-3	Complies	

Electromagnetic immunity			
The RadiForce series has been tested at the following compliance levels (C) according to the testing requirements (T) for Professional healthcare facility environments defined in IEC60601-1-2. The customer or the user of the RadiForce series should assure that it is used in such an environment.			
Immunity test	Test level (T)	Compliance level (C)	Electromagnetic environment - Guidance
Electrostatic discharge (ESD) IEC61000-4-2	±8 kV contact discharge ±15 kV air discharge	±8 kV contact discharge ±15 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transients / bursts IEC61000-4-4	±2 kV power lines ±1 kV input / output lines	±2 kV power lines ±1 kV input / output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surges IEC61000-4-5	±1 kV line to line ±2 kV line to ground	±1 kV line to line ±2 kV line to ground	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	0 % U_T (100 % dip in U_T) 0.5 cycles and 1 cycle 70 % U_T (30 % dip in U_T) 25 cycles at 50 Hz 0 % U_T (100 % dip in U_T) 250 cycles at 50 Hz	0 % U_T (100 % dip in U_T) 0.5 cycles and 1 cycle 70 % U_T (30 % dip in U_T) 25 cycles at 50 Hz 0 % U_T (100 % dip in U_T) 250 cycles at 50 Hz	Mains power quality should be that of a typical commercial or hospital environment. If the user of the RadiForce series requires continued operation during power mains interruptions, it is recommended that the RadiForce series be powered from an uninterruptible power supply or a battery.
Power frequency magnetic fields IEC61000-4-8	30 A/m (50 / 60 Hz)	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. The product should be kept at least 15 cm away from the source of power frequency magnetic fields during use.

Immunity test		Test level (T)	Compliance level (C)	Electromagnetic environment - Guidance
<p>The RadiForce series has been tested at the following compliance levels (C) according to the testing requirements (T) for Professional healthcare facility environments defined in IEC60601-1-2.</p> <p>The customer or the user of the RadiForce series should assure that it is used in such an environment.</p>				
Conducted disturbances induced by RF fields IEC61000-4-6		3 Vrms 150 kHz - 80 MHz	3 Vrms	<p>Portable and mobile RF communications equipment should be used no closer to any part of the RadiForce series, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1.2\sqrt{P}$ <p>$d = 1.2\sqrt{P}$</p> <p>$d = 1.2\sqrt{P}$, 80 MHz - 800 MHz $d = 2.3\sqrt{P}$, 800 MHz - 2.7 GHz</p> <p>Where "P" is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and "d" is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^{b)}, should be less than the compliance level in each frequency range ^{c)}.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol.</p> 
Radiated RF fields IEC61000-4-3		6 Vrms ISM ^{a)} bands between 150 kHz and 80 MHz	6 Vrms	
		3 V/m 80 MHz - 2.7 GHz	3 V/m	
Note 1	U _T is the a.c. mains voltage prior to application of the test level.			
Note 2	At 80 MHz and 800 MHz, the higher frequency range applies.			
Note 3	These guidelines regarding conducted disturbances induced by RF fields or radiated RF fields may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a)	The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6.765 MHz to 6.795 MHz, 13.553 MHz to 13.567 MHz, 26.957 MHz to 27.283 MHz, and 40.66 MHz to 40.70 MHz.			
b)	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the RadiForce series is used exceeds the applicable RF compliance level above, the RadiForce series should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the RadiForce series.			
c)	Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distances between portable or mobile RF communication equipment and the RadiForce Series

The RadiForce series is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the RadiForce series can help prevent electromagnetic interference by maintaining a minimum distance (30 cm) between portable and mobile RF communications equipment (transmitters) and the RadiForce series.

The RadiForce series has been tested at the following compliance levels (C) according to the testing requirements (T) of immunity to proximity fields from the following RF wireless communication services.

Test frequency (MHz)	Bandwidth ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Test level (T) ^{c)} (V/m)	Compliance level (C) (V/m)
385	380 - 390	TETRA 400	Pulse modulation ^{b)} 18 Hz	27	27
450	430 - 470	GMRS 460, FRS 460	FM ±5 kHz deviation 1 kHz sine	28	28
710	704 - 787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	9	9
745					
780					
810	800 - 960	GSM 800 / 900, TETRA 800, iDEN 820 CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	28	28
870					
930					
1720	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	28	28
1845					
1970					
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	28	28
5240	5100 - 5800	WLAN 802.11 a/n	Pulse modulation ^{b)} 217 Hz	9	9
5500					
5785					
a)	For some services, only the uplink frequencies are included.				
b)	The carrier is modulated using a 50 % duty cycle square wave signal.				
c)	The test levels were calculated using the maximum power, separation distance of 30 cm.				

The customer or the user of the RadiForce series can help prevent interference from proximity magnetic fields by maintaining the minimum distance (15 cm) between RF transmitters and the RadiForce series. The RadiForce series has been tested at the following conformance levels (C) according to the testing requirements (T) of immunity to proximity magnetic fields in the table below.

Test frequency	Modulation ^{a)}	Test level (T) (A/m)	Compliance level (C) (A/m)
134.2 kHz	Pulse modulation ^{a)} 2.1 kHz	65	65
13.56 MHz	Pulse modulation ^{a)} 50 kHz	7.5	7.5
a)	The carrier is modulated using a 50 % duty cycle square wave signal.		

For other portable and mobile RF communication equipments (transmitters), minimum distance between portable and mobile RF communications equipment (transmitters) and the RadiForce series as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance “d” in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where “P” is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1	At 80 MHz and 800 MHz, the separation distance for a higher frequency range applies.
Note 2	These guidelines regarding conducted disturbances induced by RF fields or radiated RF fields may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

FCC Declaration of Conformity

For U.S.A. , Canada, etc. (rated 100-120 Vac) Only

FCC Declaration of Conformity

We, the Responsible Party

EIZO Inc.

5710 Warland Drive, Cypress, CA 90630

Phone: (562) 431-5011

declare that the product

Trade name: EIZO

Model: RadiForce MX215

is in conformity with Part 15 of the FCC Rules. Operation of this product is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures.

- * Reorient or relocate the receiving antenna.
- * Increase the separation between the equipment and receiver.
- * Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- * Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note

Use the attached specified cable below or EIZO signal cable with this monitor so as to keep interference within the limits of a Class B digital device.

- AC Cord
- Shielded Signal Cable (enclosed)

Canadian Notice

This Class B digital apparatus complies with Canadian ICES-003.

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LIMITED WARRANTY

LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that (i) the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual attached to the Product (hereinafter referred to as the “**User’s Manual**”), or (ii) the LCD panel and brightness of the Product cannot maintain the recommended brightness specified in the User’s Manual in the course of normal use of the Product in accordance with the description of the User’s Manual.

The period of this Warranty is five (5) years from the date of purchase of the Product subject to the usage time being less than or equal to 30,000 hours (hereinafter referred to as the “**Warranty Period**”), provided that the brightness of the Product shall be warranted only if the Product has been used within the recommended brightness described in the User’s Manual and the Warranty Period for the brightness is limited to five (5) years from the date of purchase of the Product subject to the usage time being less than or equal to 10,000 hours (the brightness is 180 cd/m² and color temperature is 7500K). EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts of the Product upon expiration of seven (7) years after the production of such parts is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor;
- (d) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.);
- (e) Any deterioration of the Product caused by the use at higher brightness than the recommended brightness described in the User’s Manual;
- (f) Any defect of the Product caused by external equipment;
- (g) Any defect of the Product on which the original serial number has been altered or removed;
- (h) Any normal deterioration of the product, particularly that of consumables, accessories, and/or attachments (e.g. buttons, rotating parts, cables, User’s Manual, etc.); and
- (i) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

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Recycling Information

Recycling Information

This product, when disposed of, is supposed to be collected and recycled according to your country's legislation to reduce environmental burden. When you dispose of this product, please contact a distributor or an affiliate in your country.

The contact addressees are listed on the EIZO website below.

<http://www.eizoglobal.com>

For recycling information for customers in Switzerland, please refer to the following website.

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EIZO GmbH EC REP
EIZO Limited UK Responsible Person
EIZO AG CH REP